


# No.1 IN THE WORLD IN TERMS OF EFFICIENCY

## Vaginal Veil Collector V-Veil UP2™

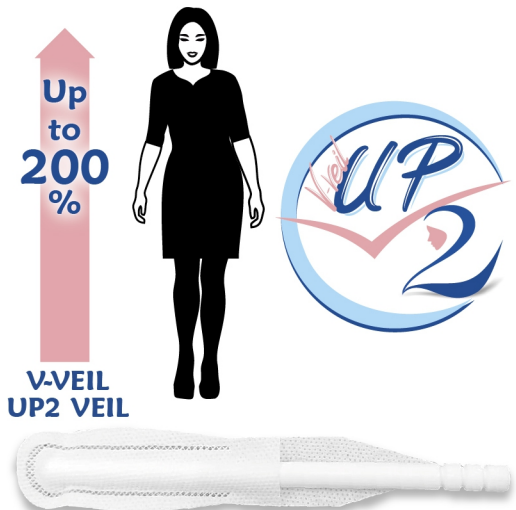

### Comparison of effectiveness between two methods





Almost twice the ability to detect high-risk oncogenic papillomaviruses (HPV-HR) by multiplex real-time PCR, using the V-Veil UP2 Vaginal Veil Collector, compared to the standard swab sampling performed by a gynaecologist or a nurse.

# Instructions for use of V-Veil UP2

Follow the instructions below before doing this test:

- Wash your hands properly before using the test
- Do not use if you have had sexual activities more recently than 2 days
- Do not use if you have washed the vaginal cavity more recently than 48 hours
- If you are pregnant or you have blood in the vaginal cavity use this medical device only at the recommendation of your doctor.

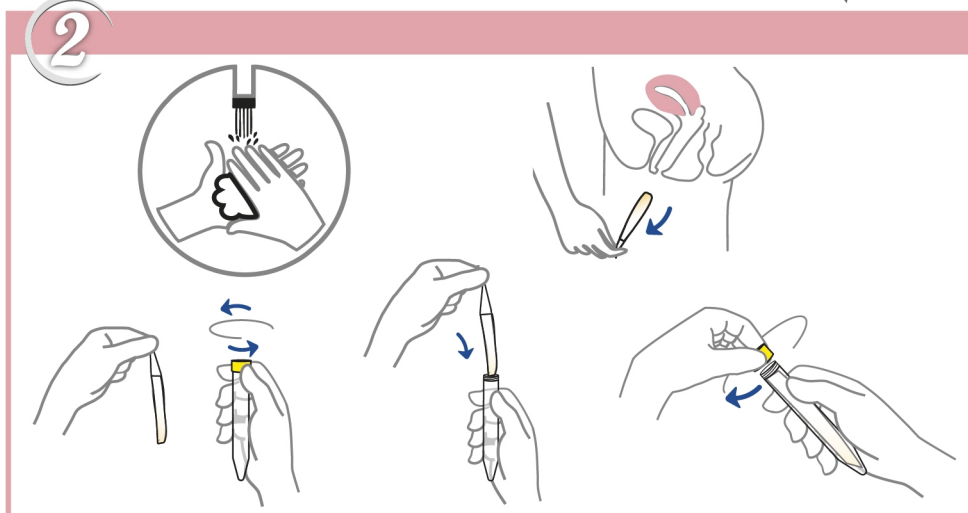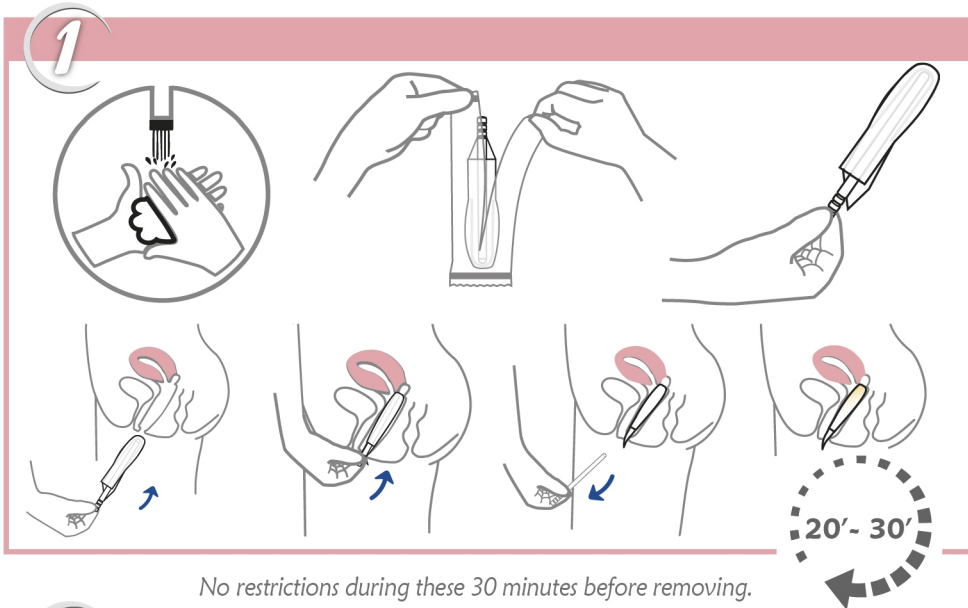


# Vaginal Veil Collector

Patent pending




**Advantage of flaps collector:** our special flaps design collect and retain more cells from the cervix area.

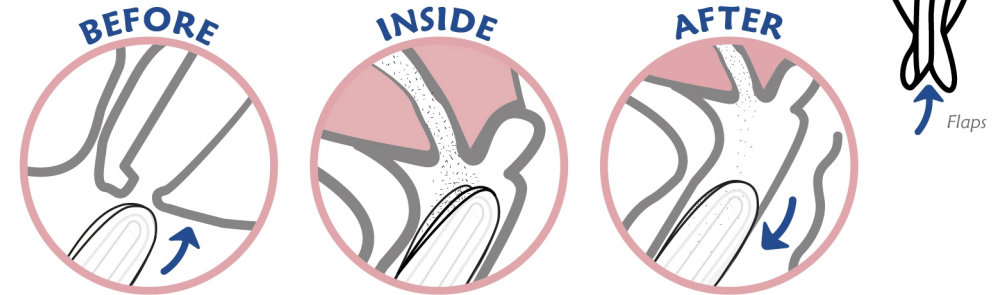

## STUDY

The graphic below compares the molecular detection of HPV genotypes performed from:

- Genital secretions self-collected by women themselves, using the Vaginal Veil Collector V-Veil Up2
- Genital secretions collected by a gynaecologist or a nurse, using a swab

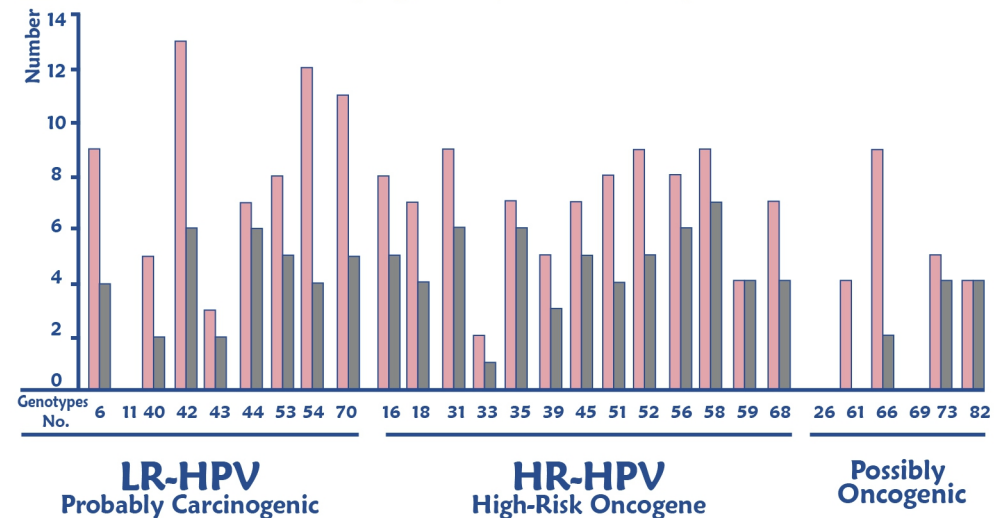

This study was performed on 256 adult women by the research team of the Laboratory of Virology of the Hospital Georges Pompidou in France. The veil detects 12 times more oncogenic HPV-HR than the use of a standard swab performed by a gynaecologist. (Details on study)

# No.1 IN THE WORLD IN TERMS OF EFFICIENCY

Vaginal Veil  
Collector

حجاب مهپلی جامع

Copyright VVeil-Up

## V-Veil UP2

14 mm Veil

14 mm



Box of 1 unit

Adapted for Ø 15 mm storage container

Box of 125 units



x1

x1



+



UV-C Disinfected



x125

x1



+



UV-C Disinfected

## V-Veil UP2

28 mm Veil

28 mm



Box of 1 unit

Adapted for Ø 30 mm storage container

Box of 125 units



x1


x1



+



UV-C Disinfected

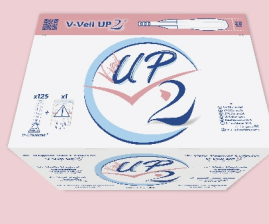

x125

x1



+




UV-C Disinfected

## UP2 KIT

28 mm Veil

28 mm

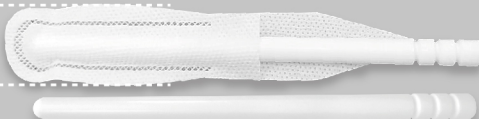

Box of 1 KIT UP2



x1

x1

x1



+



+



UV-C Disinfected UV-C Disinfected

Adapted for Ø 15mm storage container  
thanks to the 2nd applicator  
serving as a pusher

Produced in an **EN ISO 13485:2016** certified facility. Registered at European database on medical devices (EUDAMED)  
Authorized by National Agency for Medicines and Medical Devices of Romania

**CE** Medical  
Device  
CLASS I

**FDA** Approval  
soon

For more information, visit [hpv-veil.com](http://hpv-veil.com)  
Sales Department: [up2@vvup.org](mailto:up2@vvup.org)