
V-Veil UP2™

14/28 mm Veil ; Voile; Voal; Schleier; Velo; Velo; 14/28 حجاب؛ Tül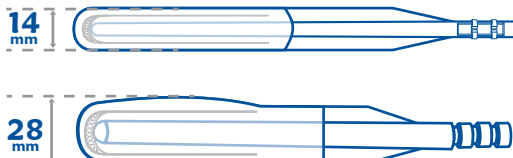

EN Vaginal Veil Collector V-Veil UP2™

Instructions
Please read and keep this leaflet

WHAT IT IS AND WHEN CAN "V-Veil UP2" BE USED?
Vaginal Veil Collector "V-Veil UP2" is a class I, non-sterile, UVC disinfected medical device for human use used to collect secretions, various fluids and substances, which are in the vaginal cavity and cervix for the diagnosis by laboratory tests. This device may be used during bleeding or pregnancy only at the recommendation of a medical professional. "V-Veil UP2" consists of veil and applicator.

HOW TO USE "V-Veil UP2"?
Wash your hands and dry them with a clean piece of cloth. Open "V-Veil UP2" pack through the intended easy opening area. Discard the used pouch with your normal household waste. Place your fingers at the end of the applicator, the grip area. Relax and choose the position which is the most comfortable for you, sitting with the legs spread, standing with one leg raised or standing with the legs slightly flexed. Spread the labia located at the opening of the vagina. Gently insert "V-Veil UP2" veil with its applicator. Keep on pushing the veil and the applicator to the grip area. The "V"-shaped terminal part should hang freely on the outside of the vagina. After inserting the Veil, remove its applicator from the vagina. To remove the applicator, gently pull it out, taking care not to pull the veil, also. Discard the applicator with your normal household waste.

CAUTION: Depending on the size of the vagina and the way you have inserted the "V-Veil UP2" in intra-vaginal veil, it is mandatory that the "V"-shaped terminal part remains visible so that it can be removed after use.

HOW TO WITHDRAW "V-Veil UP2"?
Wash your hands and dry them with a clean piece of cloth. Relax and choose the position that suits you (sitting with legs apart, standing with one leg or standing with legs bent). The veil is easily removed by pulling on "V"-shaped terminal part. After removal, place the impregnated veil in a storage container recommended by your physician or pharmacist.

- RECOMMENDATIONS AND ADVICES:**
- Wash your hands properly before using this medical device.
 - Do not use if you have had sexual activities more recently than 2 days.
 - Do not use if you have washed the vaginal cavity more recently than 48 hours.
 - If you are pregnant or you have blood in the vaginal cavity use this medical device only at the recommendation of your doctor.
 - The veil cannot and must not remain in the vagina for more than 30 minutes, but if you forget to remove it, remove it as soon as you remember.
 - Use only one "V-Veil UP2" at a time.
 - Do not reuse "V-Veil UP2", even if it is not impregnated with bodily fluids.
 - The storage container is not included. Use the storage container recommended by your physician or pharmacist.
 - For more recommendations ask your doctor or your gynecologist or your chemist.

TO WHOM IS THIS PRODUCT STRICTLY PROHIBITED?
To women who:

- have fever or flu-like symptoms, even if these are mild.
- have already suffered from the toxic shock symptom.
- have an allergy to plastics and derivatives.
- are under 15 years, without their parents and/or medical staff's consent.
- have not read the instructions included in the packaging.

THE TESTS PERFORMED ON "V-Veil UP2":

- Cytotoxicity
- Sensitisation
- Vaginal irritation (over 5 consecutive days)
- The absorption capacity of the veil (1.4 g)

RAW MATERIAL:

- The veil: Non-woven 100% Polyethylene/polyester-bico
- The applicator: 100% Polypropylene-Homopolymer for Medical & Pharmaceutical use by injection molding

IMPORTANT INFORMATION ON THE TOXIC SHOCK SYNDROME:

The Toxic Shock Syndrome (TSS) is a rare but potentially highly dangerous infection. It can occur in children, men and women (having their periods or not). It is caused by a toxin produced by a bacteria (Staphylococcus) commonly found in the human body. Half of the cases of TSS known to date occurred among women and teenagers having their monthly periods. TSS was associated with the use of tampons but it can also occur during the periods when one uses other means of protection (sanitary pads, panty-liners etc.). TSS is an infection which can be fatal, this is why it must be identified very early and quickly cured. TSS symptoms can appear suddenly during the periods or a little time afterwards. They might not all appear simultaneously: sudden fever, vomiting, diarrhoea, sore throat, dizziness, fainting, muscle pain, rash that resembles a sunburn (the skin can peel a few days later). If you show these symptoms, you must withdraw "V-Veil UP2", consult your doctor or the nearest available doctor, indicating that you have your monthly periods. TSS can happen with or without the use of "V-Veil UP2". If you already had a TSS, ask your doctor if you can use "V-Veil UP2". If you use "V-Veil UP2" in an adequate way and respect these recommendations, you decrease the risk to develop a Toxic Shock Syndrome.

STORAGE:
To be stored in clean and dry place, away from heat.

REPORTING OF INCIDENTS DUE TO THE MEDICAL DEVICE

Please note, only the leaflet is a reference for legal information as well as its indications and restrictions. Do not use the product if the "V-Veil UP2" packaging is damaged and if the expiration date is exceeded. If, despite all the care we take in the manufacture of our products, you believe that you have a defective product, please do not use it and contact the consumer service or the distributor, indicating precisely the problem you have encountered as well as the lot number specified on the packaging. Incidents related to the device can be reported to the manufacturer at: client@vvup.org or to the competent authority of the Member State where you are located. The "V-Veil UP2" Class I medical device is manufactured in the EU in accordance with the applicable provisions and requirements of REGULATION (EU) 2017/745 and complies with the requirements of standard EN ISO 13485: 2016.

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