


No.1 IN THE WORLD IN TERMS OF EFFICIENCY

Vaginal Veil Collector V-Veil UP2™



Patents pending

Comparison of effectiveness between two methods

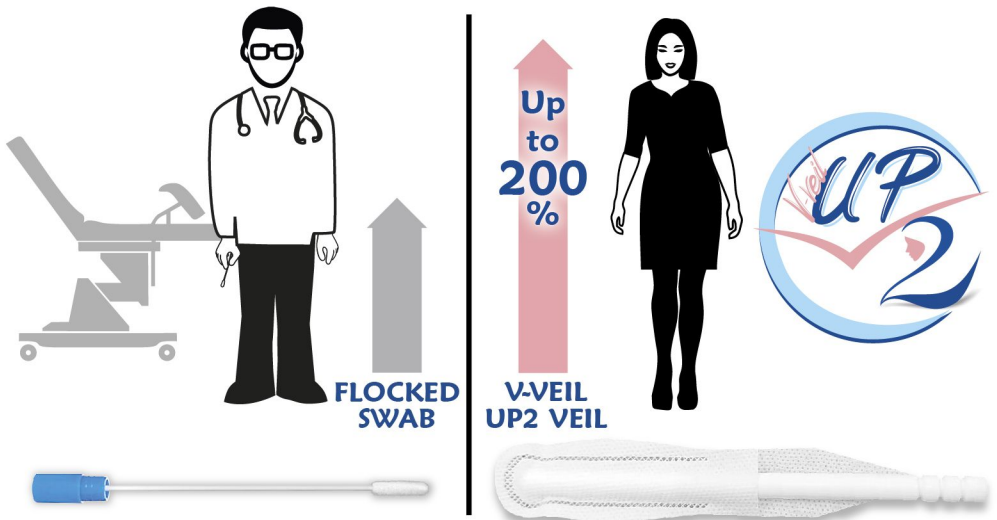


Almost twice the ability to detect high-risk oncogenic papillomaviruses (HPV-HR) by multiplex real-time PCR, using the V-Veil UP2 Vaginal Veil Collector, compared to the standard swab sampling performed by a gynaecologist or a nurse.


— Instructions for use of V-Veil UP2 —

Follow the instructions below before using this medical device::

- Wash your hands properly before using this medical device
- Do not use if you have had sexual activities more recently than 2 days
- Do not use if you have washed the vaginal cavity more recently than 48 hours
- If you are pregnant or you have blood in the vaginal cavity use this medical device only at the recommendation of your doctor.

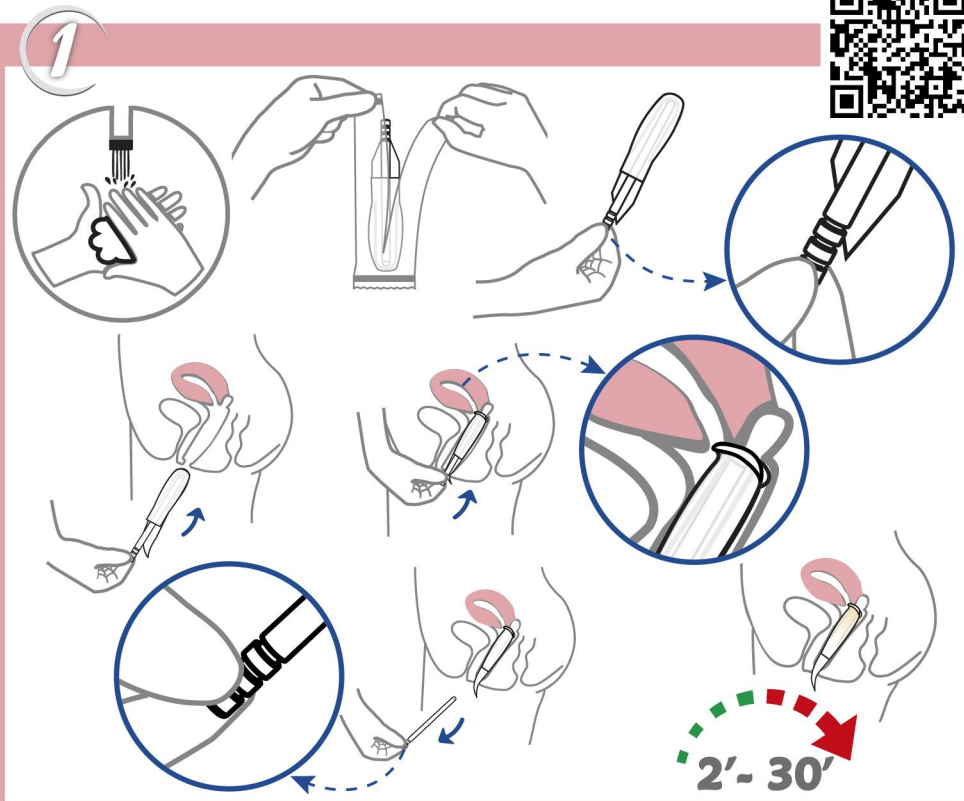

No restrictions during these 2 to 30 minutes before removing.


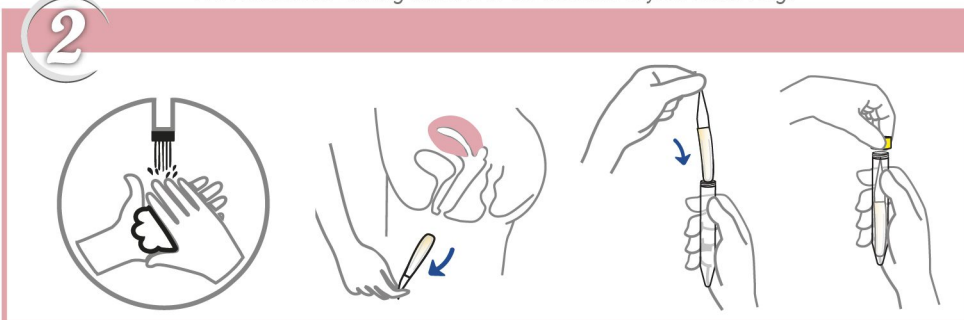


Vaginal Veil Collector

Patent pending




Advantage of flaps collector: our special flaps design collect and retain more cells from the cervix area.

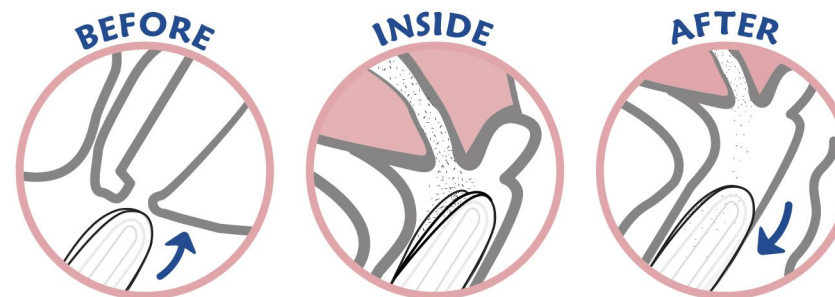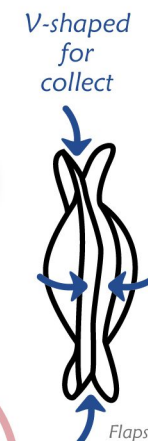

STUDY

The graphic below compares the molecular detection of HPV genotypes performed from:

- Genital secretions self-collected by women themselves, using the Vaginal Veil Collector V-Veil Up2
 Genital secretions collected by a gynaecologist or a nurse, using a flocked swab

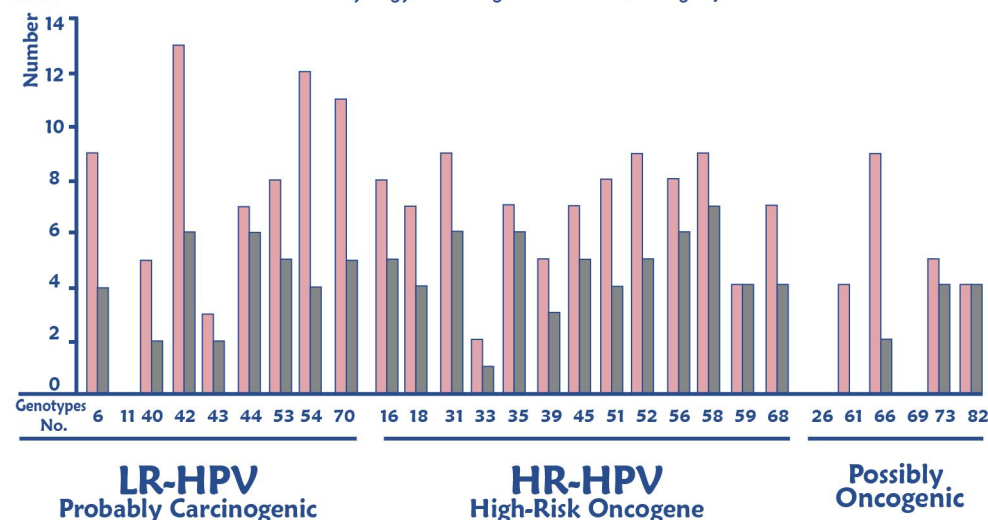

This study was performed on 256 adult women by the research team of the Laboratory of Virology of the Hospital Georges Pompidou in France. The veil detects 12 times more oncogenic HPV-HR than the use of a standard swab performed by a gynaecologist. (Details on study)

No.1 IN THE WORLD IN TERMS OF EFFICIENCY

Vaginal Veil
Collector

حجاب مهپلی جامع

Copyright V-Veil-UP

V-Veil UP2 14 mm Veil

14
mm



Adapted for Ø 15 mm storage container

Box of 1 unit

Box of 250 units



x1


x1



+



NON
STERILE
UV-C Disinfected

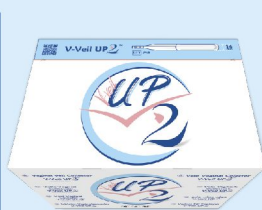

x250

x1



+



NON
STERILE
UV-C Disinfected

V-Veil UP2 28 mm Veil

28
mm



Adapted for Ø 30 mm storage container

Box of 1 unit

Box of 200 units



x1

x1



+



NON
STERILE
UV-C Disinfected



x200

x1



+



NON
STERILE
UV-C Disinfected

Produced in an EN ISO 13485:2016 certified facility. Registered at European database on medical devices (EUDAMED)
Registered in the National Agency of Medicines and Medical Devices of Romania.

MD Class I
In accordance with
MDR 2017/745

CE

FDA Approval
soon

For more information, visit hvp-veil.com
Sales Department: up2@vvup.org