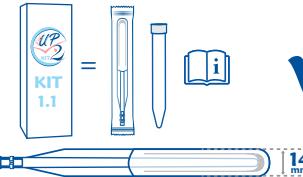
**KIT 1.1****V-Veil UP<sup>2</sup>™****KIT 1.2**

Please read and keep this leaflet  
**WHAT IS AND WHEN CAN THE "V-Veil UP<sup>2</sup>" KIT BE USED?**

The V-Veil UP<sup>2</sup> Vaginal Collector Veil KIT is used to collect secretions, various fluids and substances, from the vaginal cavity and the cervix for diagnostic laboratory testing.

**ATTENTION!** This KIT can be used during bleeding, pregnancy or following toxic shock syndrome only on the recommendation of a medical professional.

- CONTENT:**
- 1 "V-Veil UP<sup>2</sup>" Vaginal Collector Veil, which is a UVC disinfected, non-sterile, Class I medical device
  - 1 Instructions for use
  - 1 Container for the storage and transport of samples
- A stopwatch is required and is not included in the kit.

**RECOMMENDATIONS AND ADVICE:**

- Wash your hands before using this KIT.
- Do not use if you have had sexual activity in the last 2 days.
- Do not use if you have washed the vaginal cavity more recently than 48 hours.
- If you are pregnant or have bled in the vaginal cavity, use this medical device only on the advice of your doctor.
- The veil should not remain in the vagina for more than 30 minutes, but if you forget to remove it, remove it as soon as you remember.
- Use only one veil at a time.
- The veil should not be reused, even when not impregnated.
- For more recommendations, talk to your doctor, gynaecologist or pharmacist.

**WHO IS PROHIBITED FROM USING THIS PRODUCT?**

- Women who:**
- have a fever or flu-like symptoms, even mild ones,
  - have already suffered a toxic shock syndrome and do not have the recommendation from a medical specialist for taking the test,
  - are allergic to plastics and their derivatives,
  - are under 18 years of age, without parental and/or medical consent,
  - deliberately did not read the instruction sheet,
  - have bleeding and do not have a recommendation from a medical professional for taking the test,
  - are pregnant and do not have a recommendation from a medical professional for taking the test.

**IMPORTANT INFORMATION ON TOXIC SHOCK SYNDROME:** Toxic Shock Syndrome (TSS) is a rare infection that can affect children, men and women (whether they are menstruating or not). It is caused by a toxin produced by a bacterium (*staphylococcus*) that commonly exists in the body. Half of the cases of TSS recorded to date have occurred in adolescents and women during menstruation. The syndrome has been associated with the use of tampons, but it can also occur when other means of protection are used (pads, liners...). TSS is a potentially fatal infection and for this reason must be detected very early and treated immediately. Symptoms of TSS may appear suddenly, during menstruation, or shortly afterwards. They may not all be present and may not occur at the same time: sudden fever (>39°C), vomiting, diarrhea, sore throat, dizziness, fainting, muscle aches, sunburn-like rash (skin may peal a few days later). If you experience these symptoms, you should immediately remove the veil and consult your personal physician or nearest doctor. TSS can occur with or without the use of "V-Veil UP<sup>2</sup>". If you have already developed a TSS, talk to your doctor before using "V-Veil UP<sup>2</sup>". By using "V-Veil UP<sup>2</sup>" correctly and following the recommendations, you will reduce the risk of developing toxic shock syndrome.

**STORAGE:**  
Store in a clean, dry place away from heat sources.

**INCIDENT REPORTING**

**ATTENTION!** Only the instructions for use may be used as a reference for legal information, indications and restrictions. Do not use the product if the packaging is damaged and if the expiry date has passed. If, despite the measures we take in the manufacturing process, you believe you have received a defective product, please do not use it and contact the manufacturer or distributor, clearly indicating the problem and the lot number specified on the packaging. Incidents related to the device can be reported to the manufacturer at [client@vvup.org](mailto:client@vvup.org) and to the competent authority of the Member State where you are located. The "V-Veil UP<sup>2</sup>" KIT is produced in accordance with the applicable provisions and requirements of the European Union.

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