KIT 4.2

V-Veil UP2

INSTRUCTIONS FOR USE OF "V-Veil UP2" KIT

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The V-Veil UP2 Vaginal Collector Veil KIT is used to collect secretions, various fluids and substances...

ATTENTION! This KIT can be used during bleeding, pregnancy or following toxic shock syndrome...

- 1 "V-Veil UP2" Vaginal Collector Veil, which is a UVC disinfected, non-sterile, Class I medical device...

RECOMMENDATIONS AND ADVICE

- Wash your hands before using the device.
- Do not use if you have had sexual activity in the last 2 days.
- Do not use if you have washed the vaginal cavity more than 48 hours.

WHO IS PROHIBITED FROM USING THIS PRODUCT?

- Women who: have a fever or flu-like symptoms, even mild ones; have already suffered a toxic shock syndrome and do not have the recommendation from a medical specialist for taking the test.

IMPORTANT INFORMATION ON TOXIC SHOCK SYNDROME

Toxic Shock Syndrome (TSS) is a rare infection that can affect children, men and women (whether they are menstruating or not). It is caused by a toxin produced by a bacterium (Staphylococcus) that, commonly exists in the body. Half of the cases of TSS reported to date have occurred in adolescents and women during menstruation.

STORAGE:

Store in a clean, dry place away from heat sources.

INCIDENT REPORTING

ATTENTION! Only the instructions for use may be used as a reference for legal information, indications and restrictions.

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